

Dedicated to Delivery
Nasco
Life/form®

3 Life/form.
Year
Warranty
Simulators
& Trainers

Interactive Suture Trainers LF00890U and LF00892U Instruction Manual



Life/form® Products by Nasco

About the Simulator

This practice and demonstration suture trainer pad should be considered disposable, but with proper care and treatment, it will serve its purpose for a long period of time. The lifelike material can be easily cut and stitched. With a sharp scalpel, wounds can be cut at any desired depth or location on the pad. The pad is easy to remove. Each cut can be repeatedly sutured until skin at any one site becomes so heavily perforated that it will no longer support more suturing.

List of Components

- 1 jig
- 1 pad
- Nylon suture
- Instrument case
- Scalpel
- Tissue forceps
- 5½" scissors
- Needle holder
- Blue soft bag

Attaching the Pad onto the Jig

1. Take pad and place the right side onto jig.
2. Curl right side into jig.
3. Look at all four sides of the pad. Make sure all are secure on the jig.
4. Place jig onto arm or leg by positioning jig, then fasten tightly with hook-and-loop fasteners.

Removing Pad and Jig

1. To remove jig, release hook-and-loop fasteners and slip jig off of arm or leg.
2. To remove pad, lift pad from jig and replace with replacement pad when ready. (**See figure 1.**)



Figure 1

“All kits containing instruments are to be used for trainer purposes only. Instruments are not sterile and not intended for human use.”

Care of the Simulator

Do not use providone-iodine prep solutions. These solutions will indelibly stain the pad. Normal soil accumulated on the surface of the simulator can be removed with mild soap and lukewarm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains from simulator. Simply spray soiled area and wipe clean with soft cloth or paper towels. When not in continuous use, it is recommended to store the simulator in the soft storage case furnished, or cover it with a cloth.

Cautions

Solvents or corrosive materials will damage the simulator. Never place simulator on any kind of paper or plastic. These materials will transfer stains that are not removable. Ball-point pens will also make indelible stains.

Available Supplies

- | | |
|--------------------|---|
| LF00891U | Replacement Interactive Suture Pads (White) |
| LF00891(A)U | Replacement Interactive Suture Pads (White) — pkg. of 3 |
| LF00893U | Replacement Interactive Suture Pads (Black) |
| LF00893(A)U | Replacement Interactive Suture Pads (Black) — pkg. of 3 |
| LF09919U | Nasco Cleaner |



Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
LF00855U Male Catheterization
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00999U Pediatric Injectable Head
LF01005U First Aid Arm
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01034U Suture Practice Leg
LF01036U Spinal Injection
LF01037U Hemodialysis Practice Arm
LF01038U Episiotomy Suturing Set
LF01042U Suture Kit
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Infant Intraosseous Infusion
LF01121U Advanced IV Arm
LF01131U Venipuncture and Injection Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01143U Testicular Exam
LF01152U Male & Female Catheter
LF01155U Advanced CPR Dog
LF01162U Venatech IV Trainer
LF01174U NG Tube & Trach Skills
LF01184U Venatech IM & Sub Q
LF01193U Special Needs Baby
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer with Stand
LF03602U Adult Airway Management Manikin
LF03609U Child Airway Management Trainer with Stand
LF03616U Child **CRISIS**™ Manikin
LF03617U Deluxe Child **CRISIS**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03623U Infant Airway Management Trainer with stand
LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer Torso
LF03693U **Basic Buddy**® CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03709U Infant **CRISIS**™ Manikin
LF03720U **Baby Buddy**™ Infant CPR Manikin
LF03750U Fat Old Fred
LF03760U Airway Management/Cricoid Pressure Trainer
LF03770U Chest Tube
LF03953U **CRISIS**™ Manikin, complete
LF03955U Deluxe **CRISIS**™ Manikin
LF03956U Deluxe "Plus" **CRISIS**™ Manikin
LF03965U Adult **CRISIS**™ Auscultation Manikin
LF03966U Adult **CRISIS**™ Auscultation Manikin with ECG Simulator
LF04000U **GERI**™/**KERI**™ Manikin Series
LF04200U Adult Sternal Intraosseous Infusion
LF06001U CPR Prompt® Adult/Child Manikin
LF06012U CPR Prompt® Infant Manikin
LF06200U CPR Prompt® Keychain Rescue Aid
LF06204U CPR Prompt® Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 2010 PRINTED IN U.S.A. NP 066-11/RV 8-12